GAMMA Diamond

Warnings

- For **in vitro** diagnostic use (for use outside of the body only).
- For single use only.
- Healthcare professionals and other users testing multiple patients with this system should handle everything that has come into contact with human blood carefully to prevent transmitting infectious diseases, including sanitized objects.
- Please read this sheet and your Blood Glucose Monitoring System Owner’s Manual before you use this test strip. Use only GAMMA Diamond Test Strips with GAMMA Diamond Blood Glucose Monitoring System to obtain accurate results, and be covered by the manufacturer’s warranty.
- Results may be inaccurate when testing on patients with abnormally low blood pressure or those who are in shock.
- Low results may be inaccurate when testing on patients in hypoglycemic-hyperosmolar state, with or without ketosis.
- Please do not use GAMMA Diamond Blood Glucose Monitoring System on critically ill patients.
- Keep test strips and lancets away from small children. If swallowed, consult a doctor immediately for advice.

Intended Use

GAMMA Diamond test strips, when used together with GAMMA Diamond Blood Glucose Monitoring System, allow your blood glucose levels to be measured by yourself at home or by healthcare professionals. It uses fresh whole blood samples from the finger, and the following areas: the palm, forearm, upper arm, calf, and thigh. This system is not intended for use in the diagnosis or screening of diabetes mellitus.

About Alternative Site Testing (AST)

IMPORTANT: There are limitations for performing AST. Please read the GAMMA Diamond Blood Glucose Monitoring System Owner’s Manual and consult your doctor before you perform AST.

Alternative site testing (AST) is when individuals check their blood glucose levels using areas of the body other than the fingertip. The GAMMA Diamond test strips allow AST to be performed on sites other than the fingertip. We strongly recommend that you perform AST **ONLY** at the following times:

- During a pre-meal or fasting state (more than 2 hours since the last meal).
- Two hours or more after taking insulin.
- Two hours or more after exercise.

Do **NOT** use AST if:

- You think your blood glucose is low.
- You may not notice if you are hypoglycemic.
- Your AST results are inconsistent with the way you feel.
- You are testing for hypoglycemia.
- Your routine glucose results often fluctuate.

To obtain a blood sample from the alternative sites, please **rub the puncture site for approximately 20 seconds before following** the procedures of “Testing Your Blood Glucose”.

Limitations

- Hematocrit: The hematocrit level is limited to between 20% and 60%. Please ask your healthcare professional if you do not know your hematocrit level.
- Neonatal Use: **This test strip must not be used for the testing of newborns.**
- Metabolites: Dopamine, L-Dopa, methyldopa, uric acid, ascorbic acid, and acetaminophen at normal blood concentration do not significantly affect blood glucose test results.
- Lipemic Effects: Blood triglycerides up to 22.8 mmol/L do not affect the results significantly, but may affect results at higher levels.
- Altitude Effects: Altitudes up to 10,742 feet (3,275m) do not affect test results.

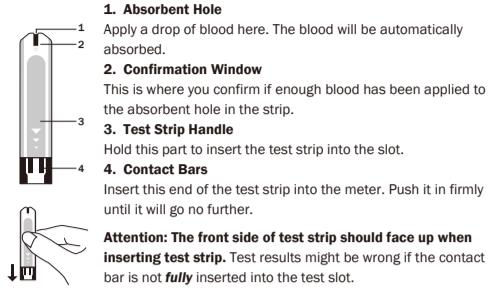
Storage and Handling

IMPORTANT: Do not use the test strips if they have expired.

- Test strips expire 3 months after first opening. Write the first opening date on the test strip vial when you first opened it.
- Store the test strips in a cool, dry place between 2°C and 32°C (35.6°F and 89.6°F) and below 85% relative humidity.
- Keep the test strips away from direct sunlight. Do not store the test strips in high humidity.
- Store the test strips in their original vial **ONLY**. Do not transfer them to a new vial or any other container.
- Do not touch the test strips with wet hands.
- Use each test strip immediately after taking it out of the vial. Close the vial immediately after taking out a strip.
- Keep the vial closed at all times.
- Do not bend, cut, or alter the test strip.

Strip Appearance

Test Strip DM

Testing Your Blood Glucose

PLEASE WASH AND DRY YOUR HANDS BEFORE PERFORMING ANY TESTS.

- Insert the test strip fully into the slot of the meter until it will go no further. When the strip is fully inserted, the meter will do several self-checks.
- Collect a blood sample with the test strip. A sufficient quantity of blood is required for the test to provide accurate results. Touch the blood drop with the absorbent hole of the test strip, and wait until the confirmation window is fully covered. **Do NOT** apply a smeared blood sample. The meter will start counting down.
- After a few seconds, the meter will display your blood glucose level. The last reading will be automatically saved in the meter. Turn it off by removing the test strip and throwing away the used test strip.

Please refer to your Owner’s Manual for more information.

The used lancet and test strip are potentially biohazardous. Please dispose of them carefully according to your local regulations.

Reading Your Result

Your blood glucose readings deliver plasma equivalent results and are displayed in millimoles of glucose per litre of blood (mmol/L).

Reference values

Time of day	Normal plasma glucose range for people without diabetes
Fasting and before meal	< 5.6 mmol/L
2 hours after meals	< 7.8 mmol/L

Source: American Diabetes Association (2010), Clinical Practice Recommendations, Diabetes Care, 33 (Supplement 1): S1-100.

Please consult your doctor to determine a target range that works best for you.

Questionable or inconsistent results

If your test results are unusual or inconsistent with how you are feeling:

- Make sure the confirmation window of the test strip is completely filled with blood.

- Check the expiry date of the test strips.
- Check the performance of your meter and test strip with the control solutions.

Please Note: Unusually high or low blood glucose levels may be symptoms of a serious medical condition. If most of your results are unusually high or low, please contact your healthcare professional.

Quality Control Testing

Our control solutions contain a known amount of glucose that can react with test strips. You can check the performance of the meter, test strip and your technique by comparing the control solution results with the range printed on the label of test strip vial. Checking regularly can ensure your test results are accurate. Please refer to the Owner’s Manual for complete testing instructions.

IMPORTANT: The reference range of the control solutions may vary with each new vial of test strips. Make sure you check the range on the label of your current vial.

Chemical Components

- > Glucose dehydrogenase FAD 8%
- > Electron shuttle 55%
- > Enzyme protector 8%
- > Non-reactive ingredients 29%

Use Only With GAMMA Diamond Blood Glucose Monitoring System

Performance Characteristics

Additional Information for Healthcare Professionals. Always wear gloves and follow your facility’s biohazard control policy and procedures when performing tests involving patient blood samples. Professionals may test with capillary and venous blood samples. Use only heparin for anticoagulation of venous blood. Home use is limited to capillary whole blood testing. Use fresh whole blood samples only.
Reaction Time: 5 seconds
Sample Size: 0.5 µL
Reaction Time: 5 seconds
System Measurement Range: 1.1 mmol/L to 33.3 mmol/L
Hematocrit Range: 20% to 60%

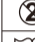
Diabetes experts have suggested that glucose meters should be within 0.83 mmol/L of the reference method when the glucose concentration is lower than 4.2 mmol/L, and be within 20% of the reference method when the glucose concentration is 4.2 mmol/L or higher. The table below displays how often GAMMA Diamond achieves this target. The chart is based on a study carried out on 160 patients to test how well GAMMA Diamond performed compared to YSI-2300 reference method results.

For glucose results < 4.2 mmol/L, the percentage (number) of GAMMA Diamond results that were within 0.83 mmol/L of the reference method.	100% (27/27)
For glucose results ≥ 4.2 mmol/L, the percentage (number) of GAMMA Diamond results were within 20% of the reference method.	100% (133/133)

*Acceptance criteria in ISO 15197: 95% of all differences in glucose values (i.e., YSI-2300 reference values minus MS’s glucose values) should be within ±0.83 mmol/L for glucose concentration < 4.2 mmol/L, and within 20% for glucose concentration ≥ 4.2 mmol/L
Note: When MS Test Strips results are compared to the reference values, difference for values below 4.2 mmol/L are expressed in mmol/L, while those above 4.2 mmol/L are in percent.

Precision
The CV (%) is less than 5% both in intermediate precision and repeatability.

Symbol Information

Symbol	Referent	Symbol	Referent
	In vitro diagnostic medical device		Batch code
	Do not use		Manufacturer
	Consult instructions for use		Serial number
	Keep away from sunlight		Caution, consult accompanying documents
	Keep dry		CE mark
	Temperature limitation		Do not use if package is damaged
	Use by / Expiry date		Green Dot: German dual waste disposal system

Made by order of: MedPack Swiss GmbH Tramstrasse 16, 9442, Bernegg, Switzerland, Tel +41 71 747 11 95, Fax +41 71 747 11 91, info@medpack-swiss.com

Producer: ForaCare Suisse AG Neugasse 55, 9000 St. Gallen, Switzerland
TDaoc Technology Corporation, 3F, 5F, No.127, Wugong 2nd Road, 24688 Wugu Township, Taipei County, TAIWAN
Distributor: LLC Dolphin Ukraine, 49000, Dnipropetrovsk, Kirova av., 107-G, tel./fax: +38 0562 31 81 04

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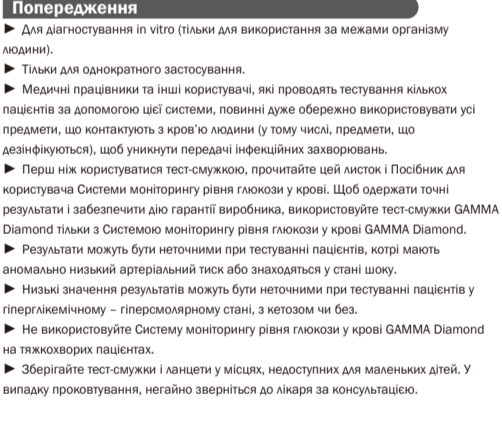
GAMMA Diamond

Test Strip DM

1. Absorbent Hole
Apply a drop of blood here. The blood will be automatically absorbed.

2. Confirmation Window

This is where you confirm if enough blood has been applied to the absorbent hole in the strip.

3. Test Strip Handle

Hold this part to insert the test strip into the slot.

4. Contact Bars

Insert this end of the test strip into the meter. Push it in firmly until it will go no further.

Attention: The front side of test strip should face up when inserting test strip. Test results might be wrong if the contact bar is not **fully** inserted into the test slot.

Testing Your Blood Glucose
PLEASE WASH AND DRY YOUR HANDS BEFORE PERFORMING ANY TESTS.

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