

*Our experience —
your gain!*

TELINJECT

Veterinärmedizinische
Spezialgeräte GmbH

G.U.T.50



Gas-Applicator for universal tele-injection of animals



TELINJECT world-wide

G.U.T.50 GAS-APPLICATOR

for universal tele-injection
of animals

The cold gas-appliator G.U.T.50 is the result of many years researching and developing.

The improved tele-injection technical standard and the experience of more than 30 years of practising immobilization of wild animals is the fundamental base of this applicator which convinces today by its wide range of application, high precision, comfortable handling and easy maintenance.

In daily practice, the tele-injection applicator is not only used for immobilization of animals but also for injections for different kind of medications such as antibiotics, vaccinations, tele-implantations of transponder for animal identification, biopsies, as well as for marking of animals.

Immobilized animals often escape before the medication is effective. Therefore it is sometimes very difficult to find them, especially in the field. A built in transmitter in the dart syringe enables to quickly find the animal by means of a special receiver.

The regulation of the gas-pressure (+/-) is continuously adjustable and is the base of an absolutely harmless tele-injection of animals. An exchange barrel 13 mm for large dart syringes enabling the injection of large volumes of medication for treatment of highly sensitive and thin skinned animals, and even birds. With the help of our applicator a large number of storks have been saved.

Due to a higher pressure volume, the G.U.T.50 can now also be used for longer distance up to 70 m on larger animals, such as elephants or rhinos. But please be always aware, that the optimal distance for a harmless tele-injection, depending from the animals size, is in the range from 5 – 50 meters.

The built in pressure gauge enables an exact pressure adjustment. The G.U.T.50 is powered either by a CO₂-device which is conveniently contained in a handle attached to the body of the applicator or by a foot pump.

The breech loading system enables a quick reloading of the dart syringe. Aiming is trough an adjustable rear sight notch which is sufficient for nearly all applications. Additionally you have the choice between several telescopic sights for longer distances or in twilight.

The G.U.T.50 is a tele-injection applicator which convinces the user even under extreme conditions. It is proven in the icy deserts of Alaska, in the tropics of Zaire, in the deserts of Saudi Arabia and also in the temperate zones of Europe and Northern America.

Technical datas:

G.U.T.50	
Weight	2,7 kg
Size (L x B x H)	1210 x 190 x 65 mm
Calibre	11 mm (Standard)/ 13 mm
Volume of syringes	1 – 3 cc (11 mm) / 5 – 20 cc (13 mm)
Pressure	0 – 16 bar
Powered by	CO ₂ -device or foot pump
Range of shooting	5 – 50 m
Aiming	by adjustable rear sight notch

Contents of delivery:

Applicator with fitting for telescopic sight and adjustable rear sight notch 800 mm standard barrel 11 mm (31L2/GUT.C) inclusive barrel cleaning set, built-in CO₂-device. Delivery in a sturdy plastic case.

Conditions of delivery:

Tele-injection devices with muzzle energy exceeding 7.5 joule are regulated by the German Weapons Act. In Germany an original copy of your valid firearms ownership license along with a Criminal Records Bureau check are required when placing an order.

Prior to ordering customers from EU countries require an import license according to: Article 11(3)/ Directive 91/477/EEC, Article 10(3)/ Directive 93/15/EWG.

In countries of the rest of the world please notice the fire arms regulation in your country.

Contact your local authorities to avoid problems during the importation of the applicator.

presented by:

Subject to alteration.

TELINJECT world-wide