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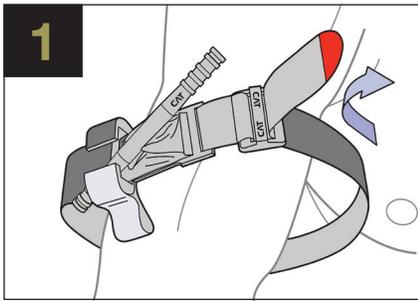
### LIMITATION OF LIABILITY:

C-A-T Resources, LLC, its parent company Composite Resources, Inc., its employees, agents, contractors, suppliers, and distributors shall assume no liability for injury or damages arising from the application or use of the Combat Application Tourniquet® (**CAT**). The user assumes all risk of liability. The **CAT** should only be used as directed by user's military service component guidelines, EMS authority, or under the supervision of a physician, and in strict compliance with these Instructions for Use.

### STANDARD USE GUIDELINES

When applied in accordance with directions, the Combat Application Tourniquet® (**CAT**) is a safe and effective device for controlling life-threatening extremity bleeding. The **CAT** has a Maximum Operating Circumference of 35in (88.9cm). The use of any tourniquet for longer than 2 hours may lead to permanent neurological or muscular damage. If you cannot be sure or cannot take the additional time to examine where the bleeding is coming from based on the situation, the **CAT** can be effectively applied over clothing as high on the arm or leg as possible. The **CAT** must NOT be applied over solid objects within the clothing. As soon as the situation permits, the injured limb should be evaluated and the **CAT** re-positioned 2"-3" above the injury directly to the skin. The **CAT** should be stored in a cool, dry place away from direct sunlight. Re-use of this single use device may introduce risk of infection. Dispose of used **CAT** in accordance with local regulations for biomedical waste. Please report all serious incidents in relation to the **CAT** to the manufacturer and appropriate regulatory authority.

Combat Application Tourniquet® and **CAT** are trademarks registered in the United States and certain other countries.

U.S. Patent Nos. 7,842,067; 7,892,253; 8,888,807; 10,016,203;  
Canada Patent No. 2,569,550;  
EU Patent Nos. 1,753,344; 2,425,786;  
Israel Patent No. 179769;  
Hong Kong Patent No. 1,163,479;  
Other Patents Pending

Manufactured by C-A-T Resources, LLC  
483 Lakeshore Pkwy, Rock Hill, SC 29730  
United States of America  
803.325.9300

NSN 6515-01-521-7976

PN 30-0001

Tactical Black

INST-001 REV02

Exclusive distribution in the U.S. by:  
North American Rescue®, LLC  
1-888-689-6277  
www.NARescue.com  
info@NARescue.com

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