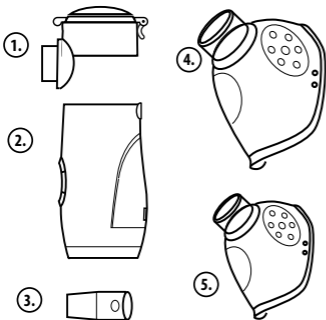
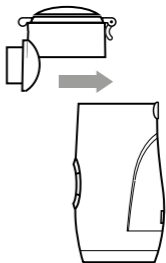
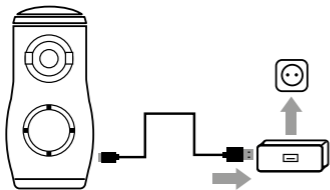
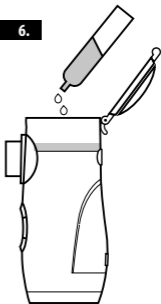
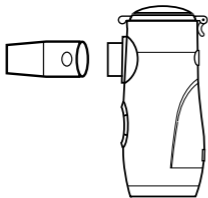
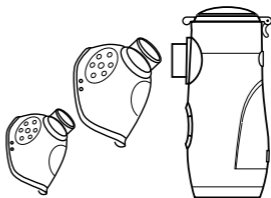
GB Mesh Nebulizer	03
PL Nebulizator siateczkowy	09
BG Нагревател за шишета с функция за стерилизиране	15
CZ Ohříváč na láhve s funkcí sterilizace	21
DE Flaschenwärmer & Sterilisator	26
EE Steriliseerimise funktsiooniga elektriline soojendi	32
GR Νεφελοποιητής πλέγματος	38
HR Električni grijač bočica i sterylizator	44
HU Üvegmelegítő fertőtlenítés funkciójával	50
LT Buteliukų šildytuvas su sterilizavimo funkcija	56
LV Pudelišu sildītājs ar sterilizēšanas funkciju	62
RO Încălzitor de biberoane cu funcție de sterilizare	68
RU Подогреватель для бутылок с функцией стерилизации	74
SK Ohrievač na fľaše s funkciou sterilizácie	80
UA Підігрівач для пляшок з функцією стерилізації	86

1.**5.****2.****6.****7.****8.**

ELECTRIC BOTTLE WARMER AND STERILIZER

Thank you for purchasing Babyono mesh nebulizer (Hereinafter referred to the device), please read the manual carefully to make sure that you can operate it correctly and safely. Please store the manual well after reading it for later reading and referencing.

Main unit 12 months for free warranty, medicine bottle (including mesh plate) 3 months for free warranty. During warranty period, we depend on the damage to decide if maintenance is required or damaged part is to be changed. Accessories are consumables, and thus do not belong to warranty range. If you need to change, we recommend our company's accessories. The details given below not including in warranty range:

- The damage caused by human fall, touch, soaking, wet
- The damage caused by not in accordance with the correct operating instructions.
- Damaged by accident.
- Without our company's authority, users who disassemble and assemble by selves.

Products Serial No. is teared or can't be recognized well, and no warranty card and No Original Invoice available.

NOTE: Take Warranty Card to contact local distributor or our company if your nebulizer needs to be repaired. Meanwhile please keep packing box in safe place so that it can be used when you send it to our company.



CHAPTER 1 - PRODUCT INTRODUCTION

1. Product overview

The device is designed for respiratory diseases treatment and respiratory tract infection prevention. The device is pushed by certain circuit frequency vibration to make piezoelectric ceramic vibrate harmony that caused high speed vibration of metal mesh. And the medicine liquid will be quickly popped through micro mesh hole of metal mesh plate to be countless micro atomizing particles. This will further transfer by inhalation treatment using masks or mouthpieces to patients' respiratory system.

2. Components (pict. 1)

1. main unit 2. medicine bottle (including atomization tube and mesh plate) 3. mouthpiece 4. adult mask, 5. child mask

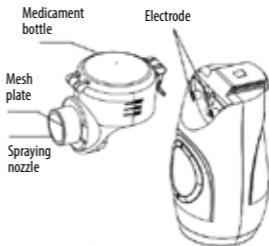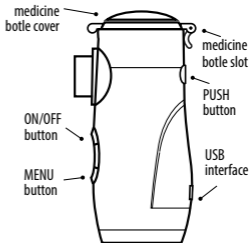3. Available Range

The nebulizer is designed for respiratory diseases treatment and respiratory tract infection prevention. It is widely used in curing diseases of respiratory system, such as cold, fever, bronchitis, asthma, pneumonia, pneumonic mycosis and pulmonary heart disease. It can also be used to cure occupational disease; give throat care to the artists, cure ENT (ear, nose, throat) diseases like chronic allergic rhinitis, a cute laryngitis, acute pharyngitis, acute tonsillitis, etc. The nebulizer shall be used with face mask, and the face mask's connection tube diameter shall be 20.0±0.1 mm, therefore the face mask can be connected to the nebulizer tightly.

4. APPLICABLE PEOPLE SCOPE

The nebulizer is available to adults, aged people, infants and children must under adults supervision. The nebulizer can be used at household environ-

ment. Contraindications:

1. The device can not be applied to anesthesia breathing system and respiratory system.
2. Infants and children must under adults supervision.
3. Pentamidine is not applicable to the device.
4. Thick or viscous liquids are not applicable to the device.

5. WORKING PRINCIPLE

The device is pushed by certain circuit frequency vibration to make piezoelectric ceramic vibrate harmony that caused high speed vibration of metal mesh. And the medicine liquid will be quickly popped through micro mesh hole of metal mesh plate to be countless micro atomizing particles. This will further transfer by inhalation treatment using masks or mouthpieces to patients' respiratory system.

6. MAIN PERFORMANCE INDEX

Please see tab. 1, it was tested by 0.9% physiological saline under the condition of temperature 25 ± 3 °C, humidity $60\% \pm 5\%$ R.H.

CHAPTER 2 - OPERATION

To prevent cross infection, the nebulizer is used only by one person. Please clean and disinfect nebulizer when you first using it or before and after using then dry it, see Chapter 3.

CHARGING THE BATTERY (PICT. 2)

1. POWER SUPPLY

This nebulizer is powered with lithium batteries. The battery can be charged with USB connect to 5Vd.c. 1.0A min. power supply with the USB cable.

- Connect the power adapter to the socket on the side of the nebulizer.
- Plug the other end of the power adapter into the wall outlet.
- The icon shines.
- When the battery is fully charged, the icon keeps on.

2. NEBULIZER ASSEMBLY

- Please clean and disinfect nebulizer before using, then dry it, see chapter 3 in details)
- Use One hand to hold the main unit, the other to hold the medicine bottle. Push follow the slot until you hear a „click“ sound (picture 5).

 **NOTE:** If not assemble medicine bottle in right place, that may caused electrode couldn't turn and nebulizer can't work.

Please keep main unit and medicine bottle clean (see chapter 3), or may cause nebulizer can't work.

1. One hand to hold the main unit, the other to open medicine bottle buckle; 2. Put medical liquid in the medicine bottle;
3. Tighten the medicine bottle cover. 4. Suggestion: max medicine liquid 10 ml, min 0.5 ml 5. After putting medicine liquid in, prevent nebulizer shaking or inverted which may cause the leakage of medicine liquid. 6. Keep medicine liquid and mesh completely contacting to prevent mesh plate air shock.

4. USAGE OF NEBULIZER

Nebulizer accessories are grouped as mouthpiece, adult mask and child mask. If need to be treated by the throat, please use the mouthpiece; meanwhile if need to be treated by nasal cavity /or throat, please choose a mask.

1. Connect mouthpiece and sending pipe of main unit as pict. 7, or connect mask and sending pipe of main unit as pict. 8;
2. Press the switch to turn on the machine
 - The digital display shows number „10“, and the icon lights up at the same time, then enter the larger atomization rate mode working (adult mode), the timing 10 minutes, and the digital display shows the value according to the countdown mode. When the time is up, its will automatically shuts down.
 - Press the menu button, the icon will light up, and the number will be displayed as „20“, then enter the smaller atomization rate mode working (Children's mode), the timing 20 minutes.
 - Press the menu button again, the icon shines, the number is displayed as „05“, then enter the cleaning mode working, the timing is 5 minutes; If you press the menu button again, its will returns to the larger atomization rate mode working, so as to cycle under the larger atomization rate mode, smaller atomization rate mode and cleaning mode, you can press the switch to turn off the machine.
 - When the medicine bottle is lack of liquid or have pure water inside, the icon C flashes 4 times and automatically shuts down.
 - If the icon blinks, means the battery voltage is low. Please charge the battery in time, If the battery is not charged in time, the icon blinks until the machine automatically shuts down.

3. Hold the nebuliser with your hand and breathe calmly and deeply:

- If you use a mouthpiece, put the flat tip into the mouth, squeeze the mouth gently so that the liquid can reach the destination.
- If you use a mask, put the mask on the face so that it covers both your nose and mouth;
- If the device is charging, the icon will flash slowly. After the device is fully charged the icon will keep bright.

NOTE:

1. Put 0.9% physiological saline into medicine bottle, after assembly and before using, press switch to test if it can work well (chapter 6).

2. Please follow pict. 7 and pict. 8, by accessories connectors to assemble mouthpiece or mask.

3. Please use correctly using mouthpiece, don't spray the liquid to your tongue.

4. Don't swallow liquid and make sure liquid does not go direct to the affected area.

5. Improper gesture and not smooth breathing may cause respiratory passage blockage.

6. Slightly tilt the Main Unit toward yourself to immerse the Vibrating Mesh in the Medicine.

7. If the medicine bottle lack of liquid or pure water inside, the device will not working

8. When the Medicine is not much, tilt the device toward yourself side, make the Medicine gathered in mesh plate, in order to make full use of the residual Medicine.

9. The device should avoid empty shock, in case of mesh plate broken.

10. Do not strongly wave or vibrate the device in use, otherwise the device automatically shut down.

5. TURN OFF THE POWER

1. Press the switch button to turn off the power.

2. When the Adapter is used, remove the power plug from the electrical outlet after using the device.

6. AFTER USING

1. Remove the mouthpiece or mask

2. Hold the Main Unit by one hand, and Press the [PUSH] button behind the Main Unit shown in pict. 9; Hold the Medicine bottle by the other hand, remove the Medicine bottle along the slot;

3. Open the lid, pour all the residual liquid medicine bottle, and promptly clean the device, allow the parts to air dry and store the device in a clean environment. See Chapters 3 and 4.

CHAPTER 3 - CLEANING AND DISINFECTING

Medicament bottle

1. Open the medicament container and discard medicament remains;

2. Pour a small amount of distilled water into the bottle and screw the cover on.

3. Press the key to activate the device. Press the menu key twice, the light will go on and „05” will be displayed in the screen. Press the automatic 5-minute cleaning function.

4. Rinse the medicament container and cover under distilled water. Collect the surplus of water with a clean cloth and let it dry off.

NOTE! Do not touch the mesh.

Main unit cleaning

1. Gently wipe the main unit with a soft and clean cloth;

2. Clean electrodes that connect the main unit to the medicament bottle in order to assure correct operation of the device.

Note: Do not immerse the main unit in water.

Mouthpiece and mask cleaning and disinfection

1. Remove the remaining amount of liquids using a gauze.

2. Clean the mouthpiece and mask with pure water for 2-3 min. (water temperature: $\leq 50^{\circ}\text{C}$);

CHAPTER 4 - MAINTENANCE

1. Do not use the device near sources of heat or naked flame. Do no dry the device and its accessories in the microwave oven, dryer, etc.

2. Make sure the nebuliser and its accessories do not contact liquids or corrosive vapours.

3. In case of defective operation of the device, please read the chapter 6 or contact the distributor.

CHAPTER 5 - SAFETY AND CARE

1. Safety precautions

- Please read the manual thoroughly and all other documents included in the box before you use the product.

- Do not shake the device while using.

- Do not repair the device and its components on your own.

- Do not touch the power adapter port or switch if your hands are wet.

- Do not immerse the main unit in water or other liquids, do not spray water or other liquids onto the main unit or power adapter.

- To avoid damage to the device, do not leave the medicament or distilled water inside the bottle.

- Dispose of the device, its components and ad-

ditional accessories as per applicable local regulations. Unlawful disposal may result in environmental pollution.

2. Care

- Do not use the medicament with esters, fat or suspended particles, including herbal medicine. It is advisable to use standard nebulisation medicaments as per the doctor's recommendations.
- Nebulisation takes 10-20 minutes, the medicament volume is 2-4 ml, but you still need to

follow the medical recommendations.

- Do not touch the mesh with hands or other items.
- Keep the device out of reach of children.
- It is advisable to keep the device away from other electronics if they consume more than 2W of power, at a distance of at least 3.3 m, in order to avoid electromagnetic interference. If you use a pacemaker, consult your physician.

CHAPTER 6 TROUBLESHOOTING

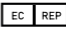
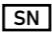
The table below shows the most common problems related to using the device and related solutions.

PROBLEM	CAUSE	SOLUTION
The device turns on but nebulisation power is low	Installation of the medicament bottle is incorrect	Install the medicament bottle in a correct way
	The medicament does not contact nebulisation parts for over 10 seconds	Position the nebuliser at a specific angle to enter the medicament
	The mesh cover of the bottle is not clean	Clean the medicament bottle, if it still malfunctions, replace with a new one
	The medicament bottle electrodes are not clean, there is some liquid or medicament left	Clean electrodes and turn on
	Medicament electrodes are not clean	Clean electrodes and turn on
The digital display does not turn on, and the device does not start nebulisation process	Installation of the medicament bottle is incorrect	Install the medicament bottle in a correct way
	The medicament has not been connected to the mesh cover in correct way	Position the nebuliser at a specific angle to enter the medicament
	Low battery	Charge the device
	The charger and device have not been connected in a correct way	Connect the device in a correct way and turn on
	The medicament bottle mesh is damaged	Before you enter the medicament, replace the medicament bottle with a new one
	The medicament bottle electrodes are not free from medicament or water	Remove medicament or water and turn the device on
	The medicament bottle electrodes are not clean	Clean electrodes and turn the device on

	The medicament bottle mesh is dirty	Clean the medicament bottle, if it still malfunctions, replace with a new one
The device shuts down while working	Installation of the medicament bottle is incorrect	Install the medicament bottle in a correct way and turn the device on
	The device has run out of medicament	Pour a suitable amount of medicament into the bottle.
	The medicament does not contact mesh for over 10 seconds	Position the nebuliser at a specific angle to enter the medicament
	Strong impacts or vibrations while working	Hold the device carefully
	The medicament bottle cannot be used	Before pouring the medicament in, replace the medicament bottle with a new one
The volume of the medicament is low and the device does not shut down automatically	Some medicaments may foam in the medicament bottle	Press the power key to turn the device off and remove foam
	There is some medicament or water on the medicament bottle electrodes	Press the power key to turn the device off and remove the medicament remains or water
	The medicament bottle or electrodes are not clean	Press the power key to turn the device off and clean electrodes
	The medicament bottle issue	Purchase a new bottle and replace

*If the above solutions prove ineffective and the device does not work properly, contact the dealer.

CHAPTER 7 MARKS AND SYMBOLS

	EC mark		Caution		Manufacturer
	Selective Collection		Environmentally friendly shelf life		EU representative
	See manual		Serial number	IP21	Waterproof rating
	BF-type application part		Date of manufacture		

ICONS AND BUTTONS

	Power key		Menu key
	Increased atomisation mode		Reduced atomisation mode
	Cleaning mode		Low liquid or water only
	Low battery		Time display

CHAPTER 8 PRODUCT SPECIFICATION

Name	Mesh nebuliser
Model	MY-135B
Applied part	Mouthpiece or mask
Dimensions	ca.52.0(L)x62.0(W)x116(H)mm
Weight (without battery)	112g
Source of power	Input: min. 5Vd.c.1.0A, Lithium battery: 3.7V,1200mAh
Battery life	When the battery is full, the product can be operated for about 80 minutes, battery life is at least 300 cycles
Safety class	BF, internally powered equipment
Software identification data	Software version; Ver. 1.0
Power consumption	Ca.2.0W
Vibration frequency	Ca.110KHz±10KHz
Nebulisation rate	≥0,2ml/min
Liquid temperature	≤40°C
Sound intensity	≤50dB for 1 meter
Particle size	MMAD<5.0µm
Capacity	Max. 10mL,min. 0.5ml
Protection against water and dust	IP21
Guarantee period	1 year for main unit (not applicable to medicament bottle)
Working temperature	Ambient temp.: 41°F~104°F/5°C, 40°C, Relative humidity:15%~90%RH, Atmospheric pressure without condensation (86.0-106.0)kPa
Storage and transportation temperature	Temperature:-20°C~55°C Relative humidity:≤80%R.H Atmospheric pressure:70.00~ 106.0 kPa
Accessories	Mouthpieces, adult mask, child mask, USB cable, operating manual

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